
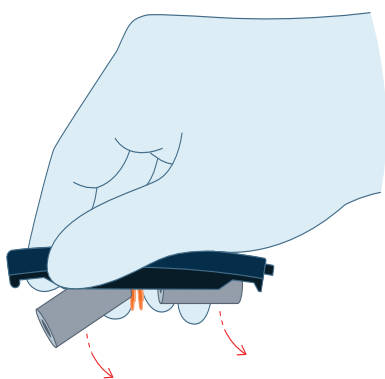


## ONETRAC LX

### TOUCH FREE BATTERY REMOVAL INSTRUCTIONS:

---

Press and pull back to remove battery cover from the ONETRAC LX handle. Batteries should remain cradled in cover and be facing upwards.

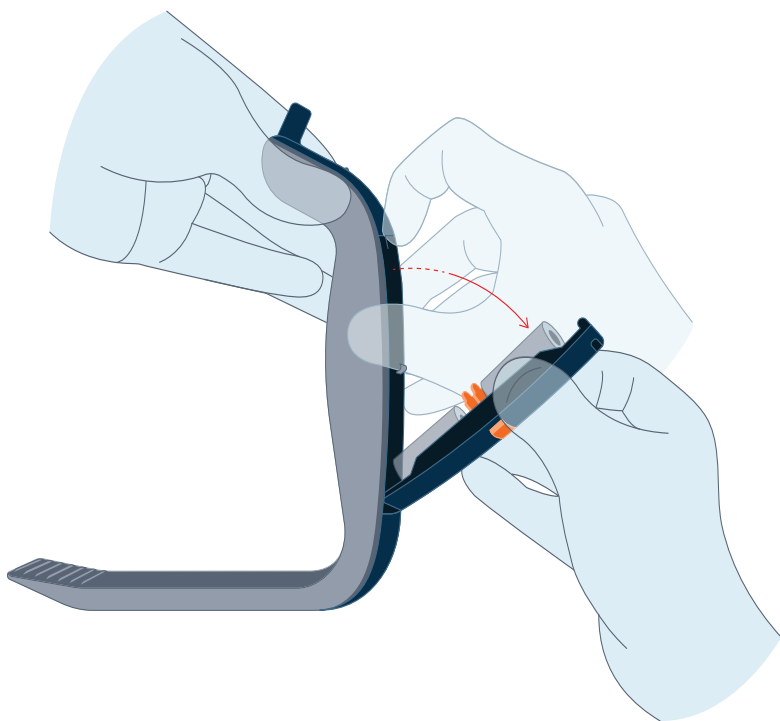

While holding battery cover over recycling bin, turn over and shake gently to release batteries from cover.



ONETRAC LX Battery Information: The primary, non-rechargeable Lithium / manganese dioxide batteries used by obp in ONETRAC LX are United States Resource Conservation and Recovery Act (RCRA) non-hazardous waste, and the brand of batteries used by obp do not contain substances regulated by California's Proposition 65. While there are no federal regulations for disposal of lithium batteries, individual states or localities can establish their own guidelines for battery disposal, and should be contacted for any disposal guidelines that they may have. Button size lithium batteries may contain perchlorate, which is regulated as a hazardous waste in California.

\*Regulations may vary in your municipality or state so please check your local regulations.